AXCENT ACRYLIC™

15 Minute Denture Base

CADMIUM FREE

Processing Instructions

Complies with ISO 20795-1:2008 Specification Type I, Class 1
Use with Garreco Axcent Acrylic 15 Minute Liquid Monomer

INTENDED USE:

For the fabrication of dental appliances.

CAUTION:

This product is to be used only under the supervision of a licensed dental practitioner.

INDICATION(S) FOR USE:

Garreco Axcent Acrylic 15 Minute Denture Base is a fast-cure acrylic resin for denture bases.

WARNINGS AND PRECAUTIONS:

IF ON SKIN: May be irritating to skin in some sensitive individuals, especially after prolonged and/or repeated contact. IF INHALED: May be irritated by gross overexposure, no matter how generated. IF SWALLOWED: Not expected to cause any harmful effects. Ingestion of large amounts may be irritating if product is swallowed, may cause nausea, headache, vomiting and/or diarrhea. IF IN EYES: May be irritating to the eyes by gross overexposure, no matter how generated. Symptoms of overexposure may include redness, itching, irritation and watering. Keep dust out of eyes.

CONTRAINDICATION(S):

Do not use with patients with a known allergy to methacrylates or other components of the product.

ADVERSE REACTION(S):

Reversible inflammatory changes of the oral mucosa may occur with susceptible persons.

STANDARD INSTRUCTIONS:

No Trial Pack Method

1. Wax, invest, and boil out the case as usual.
2. Apply Separating Film to the gypsum surfaces before mixing the acrylic.
3. Mixing and packing:
 - a. Add by volume 3 parts powder to 1 part of #1 heat cure liquid (23.4 g powder / 10 ml liquid), spatulate approximately 45 seconds. Let stand until mix is of doughy consistency (5-8 minutes). Transfer to a moist warm flask, not more than 45°C (115°F) and pat into place with fingers covered with moist cellophane or latex gloves. Be certain the flask contains an excess of material before closing.
 - b. Close flask immediately to full metal closure. Overflow should be observed on complete perimeter of flask. If overflow is noted, the flask is full and need not be reopened.
 - c. Place the flask in boiling water. Allow the water to get back to a rolling boil before starting a timer for 15 minutes.
 - d. After 15 minutes, remove the flask from the boiling water. Bench cool approximately 15-20 minutes before deflasking.

Repairs

Repairs to processed material can be made with Denture Base Repair Material, following the instructions provided.

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Garreco, LLC