



# Garreco

## ULTRA-SONIC CLEANER #1 GENERAL PURPOSE CONCENTRATE

Ultra-Sonic Cleaner #1 is a highly concentrated formula designed to remove buffing compound, food particles, pumice, rouge, tripoli and all types of soils from dentures, bridges, crowns, orthodontic appliances, aspirator tips, burs, diamonds, glass slabs, Instruments, and more. This solution can also be used as an ultra-sonic conductor when used with other ultra-sonic cleaners in a glass beaker.

—General purpose ultra-sonic cleaner

—Highly concentrated formula

**Indication for use:** Removing all types of soils from Dentures, Bridges, Crowns, Orthodontic Appliances, Aspirator Tips, Burs, Diamonds, Glass Slabs, Instruments, etc.

**Standard Size Available:** 1 Gallon (3.8 Liters)

**Directions:** Mix 1 part Ultra-Sonic Cleaner #1 with 10 parts water, clean for 2-3 minutes depending on soil. Rinse well with water.

**Item #:** 3570040



### RELATED PRODUCTS

Ultra-Sonic Cleaner #2: Tartar and Stain Remover - 3590040  
Ultra-Sonic Cleaner #5: Plaster and Stone Remover - 3610040

Manufactured by Garreco, LLC  
430 Hiram Road, Heber Springs, AR 72543, USA  
Toll Free: (800) 334-1443  
garreco.com