



Garreco

ULTRA-SONIC CLEANER #2 TARTAR AND STAIN REMOVER

Ultra-Sonic Cleaner #2 is especially formulated to easily remove calculus, food stains, permanent cement, tartar and tobacco from dentures, bridges, crowns, orthodontic appliances, and other dental restorations.

—Removes tartar and stains on dental appliances

—Fast working

Indication for use: Removes tartar and stains from dental appliances.

Standard Size Available: 1 Gallon (3.8 Liters)

Directions: Place item in covered glass beaker of undiluted Ultra-Sonic Cleaner #2. Immerse covered glass beaker in Ultra-Sonic Cleaner #1 Solution. Ultra-Sonic Cleaner #1 acts as a conductor between the tank and the beaker. **DO NOT USE ON ALUMINUM.** Cleans in 5-10 minutes. Rinse items with water after removing from Ultra-Sonic Cleaning Solution.

Item #: 3590040



RELATED PRODUCTS

Ultra-Sonic Cleaner #1: General Purpose Concentrate - 3570040

Ultra-Sonic Cleaner #5: Plaster and Stone Remover - 3610040

Manufactured by Garreco, LLC
430 Hiram Road, Heber Springs, AR 72543, USA
Toll Free: (800) 334-1443
garreco.com

Garreco, LLC is an ISO 13485 Facility
Garreco is a trademark of Garreco, LLC, Heber Springs, AR. ©2015 All rights reserved.